



OWNER'S MANUAL



THANK YOU FOR PURCHASING THE HANDY CART WHEELCHAIR ATTACHMENT.
THIS PRODUCT IS INTENDED FOR THE USE AND ENJOYMENT OF PERSONS WITH LIMITED & SEVERE MOBILITY
ASSISTANCE NEEDS THAT REQUIRE THE USE OF A WHEELCHAIR.

CONGRATULATIONS!

You have just purchased the best wheelchair attachment cart on the market today. To assure you of the best and safest performance as well as longest life, please read the enclosed information. After reading this manual, should you have a service problem or need help, please call our customer service department with the serial number of your cart.

(403) 243-2705

DAMAGE: Check the shipping carton for any visible damage. Report the damage to the freight carrier immediately. File a claim with the carrier if actual damage to your cart has occurred. Check the contents of the carton. The contents should agree with the information listed on the packing list. Report any discrepancy to the carrier and the factory immediately.

TO ORDER PARTS: Call **403-243-2705** or visit **www.jneretail.com**

TERMS: All parts will be shipped with approved credit card, or check in advance. Commercial accounts are allowed 15 day terms from date on invoice with approved credit.

FREIGHT: All freight will be paid by the customer. Special consideration will be given to items under warranty coverage.

ATTENTION: It is the responsibility of the owner to maintain the attachment cart in a safe and reliable operating condition at all times.

NOTE: Specifications found in this manual are subject to change without notice.

DAILY CARE INSTRUCTIONS FOR STORE PERSONNEL

1. CHECK ALL COMPONENTS. BE SURE TO CHECK FOR LOOSE BOLTS OR NUTS. REPLACE ANY BROKEN OR WORN PARTS.
2. CHECK ALL WARNING LABELS. REPLACE IF MISSING, WORN OR ILLEGIBLE.
3. CHECK BASKET FOR WORN AREAS THAT MAY CAUSE INJURY TO SHOPPER.
4. REPLACE RUSTED OR WORN PARTS.
5. CHAINS MUST BE CHECKED FOR WORN OR BROKEN PARTS.

DIRECTIONS

1. SECURE WHEELCHAIR BRAKES.
2. GUIDE EACH RED FORK TO A CONVENIENT PLACE ON THE CHAIR'S FRAME.
3. WRAP CHAIN AROUND FRAME AND UNDER FORK.
4. SLIDE A CHAIN LINK INTO BAR'S GROOVE TO ATTACH.
5. ENSURE WHEELCHAIR AND HANDY CART ARE SECURELY ATTACHED BEFORE USE.

INSTRUCCIONES

1. ACTIVA EL FRENO DE LA SILLA DE RUEDAS.
2. GUÍA CADA HORQUILLA ROJA AL LUGAR APROPIADO DE LA SIELLA DE RUEDAS.
3. ENVUELVE LA CADENA ALREDEDOR DEL MARCO DE LA SILLA Y DEBAJO DE LA HORQUILLA.
4. DESLICE LA CADENA EN LA RANURA DE LA BARRA PARA CONECTAR.
5. COMPRUEBA QUE LA SILLA DE RUEDA Y HANDY CART ESTÁN BIEN SUJETOS ANTES DE SU USO.



WARNING - Please read before operation. Failure to heed warnings may cause severe injury.

- NO CHILDREN ARE ALLOWED IN BASKET.
- USE INDOORS ON FLAT, LEVEL SURFACE **ONLY**. CART WILL ROLL FREELY ON INCLINE.
- ATTACHMENT CHAINS MUST BE SECURE WHEN IN USE.
- SECURE WHEELCHAIR BRAKE BEFORE ATTACHING THE HANDY CART.



ADVERTENCIAS

- LOS NIÑO NO TIENEN PERMITIDO METERSE EN EL CESTON.
- SOLAMENTE POR USO EN INTERIORES SOBRE SUPERFICIES PLANAS. CARRITO RUEDE LIBREMENTE SOBRE UN INCLINADO.
- LAS CADENAS DE FIJACION DEBEN ESTAR BIEN FIRMES MIENTRAS SE USA.
- ASEGURE LOS FRENOS DE LA SILLA DE RUEDAS ANTES DE COLOCAR EL HANDY CART.

ATTACHMENT PROCEDURE

The Handy Cart can be attached to any personal or store's wheelchair.



1. Secure wheelchair brake.



2. Guide bar to wheelchair frame for best connection.



3. Fork is now placed snugly next to chair's frame.



4. Wrap chain tightly under the fork and around frame. Slide a link into bar's groove to attach.



5. Ensure wheelchair and Handy Cart are securely attached before use.



6. Shopper is ready to go!



LIMITED WARRANTY

All **HANDY CART** parts are warranted for one year against defect, in material or workmanship. Labor is covered for **90 days** from **Date of Shipment**. Defective parts will be repaired or replaced by the company or its appointed agent. The warranty does not cover damage caused by misuse or negligence, nor does it cover defects or damages by use of unauthorized parts or service by an unauthorized person.

