

Identify Health and Identify Diagnostics USA Drug Test Cups & Dip Cards

Drugs of Abuse Assays Cross-Reactivity Manual by Generic Name

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Acetone	No known trade names	Non-reactive
Acetylsalicylic acid (aspirin)	Aspro Clear Extra Strength	Non-reactive
Acetylsalicylic acid (aspirin)	Aspro Preparations	Non-reactive
Acetylsalicylic acid (aspirin)	Bayer Aspirin	Non-reactive
Acetylsalicylic acid (aspirin)	Disprin	Non-reactive
Acetylsalicylic acid (aspirin)	Solprin	Non-reactive
Acetylsalicylic acid (aspirin)	Disprin	Non-reactive
Acetylsalicylic acid + Codeine Phosphate	Aspalgin	Positive for Opiates (OPI/MOR)
Acetylsalicylic acid + Codeine Phosphate	Codis	Positive for Opiates (OPI/MOR)
Acetylsalicylic acid + Dihydrocodeine tartrate	Codox	Positive for Opiates (OPI/MOR)
Acetylsalicylic acid + Dihydrocodeine tartrate	Disprin Forte	Positive for Opiates (OPI/MOR)
Allobarbital	No known trade names	Positive for Barbiturates (BAR)
Alphenol	No known trade names	Positive for Barbiturates (BAR)
Alprazolam	Alprax	Positive for Benzodiazepines (BZO)
Alprazolam	Alprazolam-GA	Positive for Benzodiazepines (BZO)
Alprazolam	Alprazolam Sandoz	Positive for Benzodiazepines (BZO)
Alprazolam	Genrx Alprazolam	Positive for Benzodiazepines (BZO)
Alprazolam	Kalma	Positive for Benzodiazepines (BZO)
Alprazolam	Ralozam	Positive for Benzodiazepines (BZO)
Alprazolam	Xanax	Positive for Benzodiazepines (BZO)
Albumin	Albumex 4	Non-reactive
Albumin	Albumex 20	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Amlodipine	Norvasc	Positive for Kratom
Ammonia/Ipecacuanha	Ipecac	Non-reactive
Amobarbital	Amytal	Positive for Barbiturates (BAR)
Amobarbital	Tuinal	Positive for Barbiturates (BAR)
Amoxicillin or Amoxycillin	Amoxil	Non-reactive
Amoxicillin or Amoxycillin	Amoram	Non-reactive
Amoxicillin or Amoxycillin	Augmentin	Non-reactive
Amoxicillin or Amoxycillin	Heliclear	Non-reactive
Ampicillin	Penbritin	Non-reactive
Ampicillin	Polycillin	Non-reactive
Ampicillin	Principen	Non-reactive
Antazoline Sulphate	Otrivine-Anistin	Non-reactive
Aprobarbital	No known trade names	Positive for Barbiturates (BAR)
Aspirin	Bayer Aspirin	Non-reactive
Aspirin	Excedrin	Non-reactive
Aspirin	ASA	Non-reactive
Aspirin	Angettes	Non-reactive
Aspirin	Asasantin	Non-reactive
Aspirin	Caprin	Non-reactive
Atenolol	Beta-adlat	Non-reactive
Atenolol	Co-tenidone	Non-reactive
Atenolol	Kalten	Non-reactive
Atenolol	Tenben	Non-reactive
Atenolol	Tenif	Non-reactive
Atenolol	Tenoret 50	Non-reactive
Atenolol	Tenorectic	Non-reactive
Atenolol	Tenormin	Non-reactive
Atorvastatin	Lipitor	Non-reactive
Azlocillin	Securopen	Non-reactive
Barbital	No known trade names	Positive for Barbiturates (BAR)
Beclometasone	AeroBec	Non-reactive
Beclometasone	AeroBec Forte	Non-reactive
Beclometasone	Asmabec	Non-reactive
Beclometasone	Beclazone	Non-reactive
Beclometasone	Becloforte	Non-reactive
Beclometasone	Becodisks	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Beclometasone	Beconase	Non-reactive
Beclometasone	Becotide	Non-reactive
Beclometasone	Fillair	Non-reactive
Beclometasone	Nasobec	Non-reactive
Beclometasone	Qvar	Non-reactive
Beclometasone	Ventide	Non-reactive
Beclometasone	Zonivent	Non-reactive
Bendrofluazide	Aprinox	Non-reactive
Bendrofluazide	Cogaretic	Non-reactive
Bendrofluazide	Indertic	Non-reactive
Bendrofluazide	Inderex	Non-reactive
Bendrofluazide	Neo-Naclax	Non-reactive
Bendrofluazide	Neo-Naclax-K	Non-reactive
Bendrofluazide	Prestim	Non-reactive
Bendrofluazide	Tenben	Non-reactive
Benzalkonium	Bradosol	Non-reactive
Benzylpenicillin	Crystapen	Non-reactive
Bromazepam	Lexotan	Positive for Benzodiazepines (BZO)
Brompheniramine	Dimetapp	Non-reactive
Brompheniramine	Dimotapp	Non-reactive
Brompheniramine	Dimotane	Non-reactive
Buprenorphine	Subutex	(BUP)
Buprenorphine	Temgesic	(BUP)
Bupropion	Wellbutrin	Positive for Kratom
Buspirone	Buspar	Potential positive for FEN
Butabarbital	Butisol	Positive for Barbiturates (BAR)
Butabarbital	Soneryl	Positive for Barbiturates (BAR)
Butalbital	Fioricet	Positive for Barbiturates (BAR)
Butalbital	Fiorinal	Positive for Barbiturates (BAR)
Butethal	No known trade names	Positive for Barbiturates (BAR)
Canestan	Canestan ear/skin/vaginitis/HC	Non-reactive
Carbamazepine	Tegretol	Non-reactive
Carbamazepine	Teril	Non-reactive
Carbamazepine	Timonil	Non-reactive
Cephalexin	Ceporex	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Cephalexin	Keflex	Non-reactive
Chloral Hydrate	Welldorm	Non-reactive
Chlorazepate	Tranxene	Positive for Benzodiazepines (BZO)
Chlordiazepoxide	Librium	Positive for Benzodiazepines (BZO)
Chlorhexidine Gluconate	Bactrigras	Non-reactive
Chlorhexidine Gluconate	Cordsodyl	Non-reactive
Chlorhexidine Gluconate	Chlorohex	Non-reactive
Chlorhexidine Gluconate	CX Powder	Non-reactive
Chlorhexidine Gluconate	Dermol	Non-reactive
Chlorhexidine Gluconate	Hibicet	Non-reactive
Chlorhexidine Gluconate	Hibiscrub	Non-reactive
Chlorhexidine Gluconate	Hibisol	Non-reactive
Chlorhexidine Gluconate	Hibitane	Non-reactive
Chlorhexidine Gluconate	Instillagel	Non-reactive
Chlorhexidine Gluconate	Naseptin	Non-reactive
Chlorhexidine Gluconate	Nystaform	Non-reactive
Chlorhexidine Gluconate	Serotulle	Non-reactive
Chlorhexidine Gluconate	Steripod	Non-reactive
Chlorhexidine Gluconate	Tisept	Non-reactive
Chlorhexidine Gluconate	Unisept	Non-reactive
Chlorhexidine Gluconate	Uriflex	Non-reactive
Chlorhexidine Gluconate	Uro-Trainer	Non-reactive
Chlorpheniramine Maleate	Galpseud	Non-reactive
Chlorpheniramine Maleate	Haymine	Non-reactive
Chlorpheniramine Maleate	Piriton	Non-reactive
Chlorpromazine	Largactil	Non-reactive
Cimetidine	Dyspamet	Non-reactive
Cimetidine	Tagamet	Non-reactive
Cimetidine	Zita	Non-reactive
Clobazam	Frisium	Positive for Benzodiazepines (BZO)
Clonazepam	Clonopin	Potential Negative for Benzodiazepines (BZO)-urine metabolite not detected

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Clonazepam	Klonopin	Potential Negative for Benzodiazepines (BZO)-urine metabolite not detected
Clonazepam	Rivotril	Potential Negative for Benzodiazepines (BZO)-urine metabolite not detected
Clonazolam		Potential Negative for Benzodiazepines (BZO)-urine metabolite not detected
Citalopram	Cipramil	Non-reactive
Co-Amoxiclav	Augmentin	Non-reactive
Co-Amoxiclav	Augmentin Duo	Non-reactive
Codeine Phosphate	Codafen Continus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Codeine Linctus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Pediatric BP	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Galcodine	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Kapake	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Migraleve	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Solpadol	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Tylox	Positive for Opiates (OPI/MOR)
Co-Fluampicil	Magnapen	Non-reactive
Co-Phenotrope (atropine/diphenoxylate)	Lomotil	Non-reactive
Co-Phenotrope (atropine/diphenoxylate)	Trepergen	Non-reactive
Delorazepam	Briantum	Positive for Benzodiazepines (BZO)
Dexamethasone	Decadron	Non-reactive
Dexamethasone	Dexa-Rhinaspray Duo	Non-reactive
Dexamethasone	Maxidex	Non-reactive
Dexamethasone	Maxidrol	Non-reactive
Dexamethasone	Minims	Non-reactive
Dexamethasone	Otomize	Non-reactive
Dexamethasone	Sofradex	Non-reactive
Dexamphetamine Sulphate	Adderall	Positive for Amphetamine (AMP)
Dexamphetamine Sulphate	Adderall XR	Positive for Amphetamine (AMP)
Dexamphetamine Sulphate	Dexedrine	Positive for Amphetamine (AMP)
Dextropropoxyphene	Darvon	(PPX)
Dextropropoxyphene	Darvocet	(PPX)
Dextropropoxyphene	Co-proximal	(PPX)

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Diazepam	Diazemuls	Positive for Benzodiazepines (BZO)
Diazepam	Stesolid	Positive for Benzodiazepines (BZO)
Diazepam	Valclair	Positive for Benzodiazepines (BZO)
Diazepam	Valium	Positive for Benzodiazepines (BZO)
Diclofenac Sodium	Dicloflex	Non-reactive
Diclofenac Sodium	Diclomax	Non-reactive
Diclofenac Sodium	Motifene	Non-reactive
Diclofenac Sodium	Volraman	Non-reactive
Diclofenac Sodium	Volsaid	Non-reactive
Diclofenac Sodium	Voltarol	Non-reactive
Dicyclomine	Kolanticon	Non-reactive
Dicyclomine	Merbentyl	Non-reactive
Dihydrocodeine	DHC Continus	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Paramol	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine Forte	Positive for Opiates (OPI/MOR)
Dimethicone	Asilone	Non-reactive
Dimethicone	Fancorsil	Non-reactive
Diphenhydramine	Benadryl	Non-reactive
Diphenhydramine	Medinex	Non-reactive
Diphenhydramine	Nytol	Non-reactive
Diphenhydramine	Panadol Night	Non-reactive
Diphenoxylate with Atropine	Lomotil	Non-reactive
Dothiepin	Prothiaden	Potential Positive for Tricyclic Antidepressants (TCA)
Docusate	Colace, Diocto, Doc-Q-Lace, Docu, Doculase	Positive for Kratom
Doxycycline	Vibramycin	Non-reactive
Doxycycline	Vibramycin-D	Non-reactive
Doxepin	Sinequan	Positive for Tricyclic Antidepressants (TCA)

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Doxepin	Xepin	Positive for Tricyclic Antidepressants (TCA)
Efavirenz	Sustiva	Potential False Positive for THC
Erythromycin	Arpimycin	Non-reactive
Erythromycin	Benzymycin	Non-reactive
Erythromycin	Erycane	Non-reactive
Erythromycin	Erymax	Non-reactive
Erythromycin	Erythrocin	Non-reactive
Erythromycin	Erythroped	Non-reactive
Erythromycin	Ilosone	Non-reactive
Erythromycin	Isotrexin	Non-reactive
Erythromycin	Lactobinate	Non-reactive
Erythromycin	Stiemycin	Non-reactive
Erythromycin	Tiloryth	Non-reactive
Erythromycin	Zineryl	Non-reactive
Estazolam	ProSom	Positive for Benzodiazepines (BZO)
Ethambutol	Myambutol	Non-reactive
Ethanol	all alcoholic beverages or syrups (cough syrups, etc)	Positive for urine alcohol (ALC) and Ethylglucuronide (ETG)
Ethylmorphine	No known trade names	Positive for Opiates (OPI/MOR)
Famprofazone	Gewodin, Gewolin (not US)	Positive for Methamphetamine (MET) and Amphetamine (AMP)
Flualprazolam		Positive for BZO
Flucloxacillin	Co-fluampicil	Non-reactive
Flucloxacillin	Floxapen	Non-reactive
Flucloxacillin	Magnapen	Non-reactive
Flunitrazepam	Rohypnol	Positive for Benzodiazepines (BZO)
Fluoxetine	Prozac	Non-reactive
Fluoxetine	Sarafem	Non-reactive
Flupentixol Decanoate	Depixol	Non-reactive
Flupentixol Decanoate	Fluanxol	Non-reactive
Flubromazolam		Positive for Benzodiazepines (BZO)
Fusidic Acid (Sodium Fusidate)	Fucithalmic	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Fusidic Acid (Sodium Fusidate)	Fucibet	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucidin	Non-reactive
Gabapentin	Gralise, Horizant, Neuraptine	Positive for Gabapentin (GAB)
Gentamicin Sulphate	Cidomycin	Non-reactive
Gentamicin Sulphate	Genticin	Non-reactive
Gentamicin Sulphate	Gentisone	Non-reactive
Hydrocodone	Lorcet	Positive for Opiates (OPI/MOR)
Hydrocodone	Loratab	Positive for Opiates (OPI/MOR)
Hydrocodone	Vicodin	Positive for Opiates (OPI/MOR)
Hydrocortisone	Actinac	Non-reactive
Hydrocortisone	Alphaderm	Non-reactive
Hydrocortisone	Alphosyl-HC	Non-reactive
Hydrocortisone	Anugesic-HC	Non-reactive
Hydrocortisone	Anusol-HC	Non-reactive
Hydrocortisone	Calmurid-HC	Non-reactive
Hydrocortisone	Canesten-HC	Non-reactive
Hydrocortisone	Colioam	Non-reactive
Hydrocortisone	Daktacort	Non-reactive
Hydrocortisone	Dioderm	Non-reactive
Hydrocortisone	Econacort	Non-reactive
Hydrocortisone	Efortelan	Non-reactive
Hydrocortisone	Efcortesel	Non-reactive
Hydrocortisone	Eurax-HC	Non-reactive
Hydrocortisone	Gregoderm	Non-reactive
Hydrocortisone	Hydrocortistab	Non-reactive
Hydrocortisone	Hydrocortone	Non-reactive
Hydrocortisone	Mildison lipocream	Non-reactive
Hydrocortisone	Neo-cortef	Non-reactive
Hydrocortisone	Nystaform	Non-reactive
Hydrocortisone	Perinal	Non-reactive
Hydrocortisone	Proctofoam	Non-reactive
Hydrocortisone	Quinocort	Non-reactive
Hydrocortisone	Terra-cortil	Non-reactive
Hydrocortisone	Timodine	Non-reactive
Hydrocortisone	Uniroid-HC	Non-reactive
Hydrocortisone	Vioform-HC	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Hydrocortisone	Xyloproct	Non-reactive
Hydrocortisone-17-butyrate	Locoid	Non-reactive
Hydrocortisone-17-butyrate	Locoid C	Non-reactive
Hydromorphone	Dilaudid	Positive for Opiates (OPI/MOR)
Hydromorphone	Hydrostat	Positive for Opiates (OPI/MOR)
Hydroxychloroquine	Plaquenil	Positive for GAB at high concentration
Hydroxyzine Hydrochloride	Atarax	Potential False Positive for FEN
Hydroxyzine Hydrochloride	Vistaril	Potential False Positive for FEN
Hydroxyzine Hydrochloride	Ucerax	Potential False Positive for FEN
Hyoscine butylbromide	Buscopan	Non-reactive
Hyoscine butylbromide	Scopoderm	Non-reactive
Ibuprofen	Brufen	Non-reactive
Ibuprofen	Codafen	Non-reactive
Ibuprofen	Fenbid	Non-reactive
Ibuprofen	Ibugel	Non-reactive
Ibuprofen	Ibuspray	Non-reactive
Ibuprofen	Motrin	Non-reactive
Ibuprofen	Proflex	Non-reactive
Indomethacin	Flexin continus	Non-reactive
Indomethacin	Indocid	Non-reactive
Indomethacin	Indomod	Non-reactive
Interferon Alfa	Roferon-A	Non-reactive
Interferon Alfa	Intron A	Non-reactive
Interferon Alfa	Viraferon	Non-reactive
Interferon Alfa	Wellferon	Non-reactive
Isoniazide	INH	Non-reactive
Isosorbide Dinitrate	Angitak	Non-reactive
Isosorbide Dinitrate	Cedocard	Non-reactive
Isosorbide Dinitrate	Isocard	Non-reactive
Isosorbide Dinitrate	Isoket	Non-reactive
Isosorbide Dinitrate	Isordil	Non-reactive
Isosorbide Dinitrate	Sorbichew	Non-reactive
Isosorbide Dinitrate	Sorbitrate	Non-reactive
Isosorbide Dinitrate	Sorbid SA	Non-reactive
Koalin and Morphine Mixture	Diocalm	Positive for Opiates (OPI/MOR)

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Koalin and Morphine Mixture	Entersan	Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Opazimes	Positive for Opiates (OPI/MOR)
Ketoprofen	Orudis	Non-reactive
Ketoprofen	Oruvail	Non-reactive
Ketoprofen	Powergel	Non-reactive
Lactulose	Lactugal	Non-reactive
Lactulose	Duphalec	Non-reactive
Lansoprazole	Heliclear	Non-reactive
Lansoprazole	Zoton	Non-reactive
Lisdexamphetamine	Vyvanse	Positive for Amphetamine (AMP)
Lisinopril HCl	Carace	Non-reactive
Lisinopril HCl	Zestril	Non-reactive
Lisinopril HCl	Zestoretic	Non-reactive
Lofexidine	Britoflex	Non-reactive
Loperamide HCl	Imodium	Non-reactive
Loperamide HCl	Loperagen	Non-reactive
Loperamide HCl	Norimide	Non-reactive
Loratadine	Claritin	Non-reactive
Lorazepam	Ativan	Positive for Benzodiazepines (BZO)
Lormetazepam	Noctamide	Positive for Benzodiazepines (BZO)
Magnesium Alginate	Gaviscon	Non-reactive
Mannitol	Gaviscon	Non-reactive
Medazepam	Anxitol	Positive for Benzodiazepines (BZO)
Medazepam	Lerisum	Positive for Benzodiazepines (BZO)
Medazepam	Medacepan	Positive for Benzodiazepines (BZO)
Medazepam	Nobritol	Positive for Benzodiazepines (BZO)
Medazepam	Nobrium	Positive for Benzodiazepines (BZO)
Mefenamic Acid	Ponstan	Non-reactive
Mefenamic Acid	Ponstel	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Meloxicam	Vivlodex, Mobic	Potential positive for kratom (KRA) at high concentraion
Meperidine	Demerol	Non-reactive
Meperidine	Pethidine	Non-reactive
Metoclopramide	Reglan	Non-reactive
Methadone Hydrochloride	Dolophine	Positive for Methadone (MTD) and Methadone Metabolite (EDDP)
Methadone Hydrochloride	Methadose	Positive for Methadone (MTD) and Methadone Metabolite (EDDP)
Methadone Hydrochloride	Physetone	Positive for Methadone (MTD) and Methadone Metabolite (EDDP)
d-Methamphetamine HCL	Desoxyn	Positive for Methamphetamine (MET)
d-Methamphetamine HCL	Methedrine	Positive for Methamphetamine (MET)
d-Methamphetamine HCL	Methamprex	Positive for Methamphetamine (MET)
l-Methamphetamine HCL	Vick's inhalers	Positive for Methamphetamine (MET)
Methylenedioxyamphetamine (MDA)	Eve (slang)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Methylenedioxyamphetamine (MDA)	Love Drug (slang)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	Ecstasy (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	XTC (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	Adam (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	E (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Metoprolol	Lopressor, Toprol	Positive for kratom
Metronidazole	Anabact	Non-reactive
Metronidazole	Elyzol	Non-reactive
Metronidazole	Flagyl	Non-reactive
Metronidazole	Metrogel	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Metronidazole	Metrolyl	Non-reactive
Metronidazole	Metrotop	Non-reactive
Metronidazole	Neuratop	Non-reactive
Metronidazole	Noritate	Non-reactive
Metronidazole	Rozex	Non-reactive
Metronidazole	Zidoval	Non-reactive
Metronidazole	Zymoet	Non-reactive
Mitragynine	Kratom	Positive for Kratom
Mirtazapine	Remeron, Remeron	Positive for Kratom
Morphine	Astramorph	Positive for Opiates (OPI/MOR)
Morphine	Cyclimorph	Positive for Opiates (OPI/MOR)
Morphine	Duramorph	Positive for Opiates (OPI/MOR)
Morphine	Morcap	Positive for Opiates (OPI/MOR)
Morphine	Morphine Sulfate	Positive for Opiates (OPI/MOR)
Morphine	MS Contin	Positive for Opiates (OPI/MOR)
Morphine	Oramorph	Positive for Opiates (OPI/MOR)
Morphine	Roxanol	Positive for Opiates (OPI/MOR)
Morphine	Severedol	Positive for Opiates (OPI/MOR)
Nadolol	Corgard	Non-reactive
Nadolol	Corgaretic	Non-reactive
Nadolol	Corgartic	Non-reactive
Naloxone	Narcan	Non-reactive
Naltrexone	Antaxone	Positive for Oxycodone, Kratom
Naltrexone	Nalorex	Positive for Oxycodone, Kratom
Naltrexone	Trexan	Positive for Oxycodone, Kratom
Naproxen	Aleve	Non-reactive
Naproxen	Condrotec	Non-reactive
Naproxen	Naprtec	Non-reactive
Naproxen	Naprosen	Non-reactive
Naproxen	Naprosyn	Non-reactive
Naproxen	Nycopren	Non-reactive
Naproxen	Synflex	Non-reactive
Nicotine	Nicoderm	Non-reactive
Nicotine	Nicorette	Non-reactive
Nicotine	Nicotinell	Non-reactive
Nicotine	Nicotrol	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Nicotine	Niquitin	Non-reactive
Nifedipine	Adalat	Non-reactive
Nifedipine	Nifecard	Non-reactive
Nifedipine	Nifar	Non-reactive
Nifedipine	Nif-ten	Non-reactive
Nifedipine	Procardia	Non-reactive
Nitrazepam	Mogadon	Positive for Benzodiazepines (BZO)
Nitrazepam	Somnite	Positive for Benzodiazepines (BZO)
Norethindrone	BiNovum	Non-reactive
Norethindrone	Brevinor	Non-reactive
Norethindrone	Climagest	Non-reactive
Norethindrone	Climesse	Non-reactive
Norethindrone	Elleste Duet	Non-reactive
Norethindrone	Estracombi	Non-reactive
Norethindrone	Evorel	Non-reactive
Norethindrone	Kliofem	Non-reactive
Norethindrone	Klovance	Non-reactive
Norethindrone	Loestrin	Non-reactive
Norethindrone	Micronor	Non-reactive
Norethindrone	Norlutin	Non-reactive
Norethindrone	Noriday	Non-reactive
Norethindrone	Norimin	Non-reactive
Norethindrone	Norinyl	Non-reactive
Norethindrone	Noristat	Non-reactive
Norethindrone	Nuvelle	Non-reactive
Norethindrone	Ovysmen	Non-reactive
Norethindrone	TriNovum	Non-reactive
Norethindrone	Trisquens	Non-reactive
Norethindrone	Utovian	Non-reactive
Noscapine	Narcotine	Non-reactive
d/l-Octopamine	No known trade names	Non-reactive
Omeprazole	Zegerid OTC, OmePPi, Zegerid, and Prilosec	Potential Positive for Kratom
Olanzapine	Zyprexa	Positive for Kratom

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Orphenadrine	Norflex	Non-reactive
Orphenadrine	Norgesic	Non-reactive
Orphenadrine	Disipil	Non-reactive
Oxazepam	Serax	Positive for Benzodiazepines (BZO)
Oxazepam	Ox-pam	Positive for Benzodiazepines (BZO)
Oxymetazoline	Afrin	Non-reactive
Oxymetazoline	Neo-synephrine	Non-reactive
Oxytetracycline	Terra-Cortil	Non-reactive
Oxytetracycline	Terramycin	Non-reactive
Oxytetracycline	Trimovate	Non-reactive
Paracetamol (Acetaminophen)	Fortagesic	Non-reactive
Paracetamol (Acetaminophen)	Paradote	Non-reactive
Paracetamol (Acetaminophen) /Codeine Preparations	Tylenol 3	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-codamol	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Codafen	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-dydramol	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Kapake	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Remedine	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Solpadol	Positive for Opiates (OPI/MOR)
Paracetamol (Acetaminophen) /Codeine Preparations	Tylox	Positive for Opiates (OPI/MOR)
Paroxetine	Paxil	Non-reactive
Paroxetine	Seroxat	Non-reactive
Penicillin	Combicillin	Non-reactive
Penicillin	Mefoxin	Non-reactive
Pentobarbital	Nembutal	Positive for Barbiturates (BAR)
Pericyazine	Amplan	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Pericyazine	Aolept	Non-reactive
Pericyazine	Apamin	Non-reactive
Pericyazine	Iryakin	Non-reactive
Pericyazine	Nemactil	Non-reactive
Pericyazine	Neulactil	Non-reactive
Pericyazine	Propetyl	Non-reactive
Pericyazine	Psycholept	Non-reactive
Phenobarbitone (see also Phenobarbital)	Luminal	Positive for Barbiturates (BAR)
Phenytoin	Dilantin	Non-reactive
Phenytoin	Epanutin	Non-reactive
Phenytoin	Epitard	Non-reactive
Phenobarbital	Donnatal	Positive for Barbiturates (BAR)
Pholcodine	Galenphol	Positive for Opiates (OPI/MOR)
Pholcodine	Strong BP	Positive for Opiates (OPI/MOR)
Pholcodine	Pavacol-D	Positive for Opiates (OPI/MOR)
Pholcodine	Thebacon	Positive for Opiates (OPI/MOR)
Piperacillin	Pipracil	Non-reactive
Piperacillin	Tazobactam	Non-reactive
Prazepam	Centrax	Positive for Benzodiazepines (BZO)
Prazepam	Demetrin	Positive for Benzodiazepines (BZO)
Prednisolone	Delta-cortef	Non-reactive
Prednisolone	Econopred	Non-reactive
Prednisolone	Inflamase	Non-reactive
Prednisolone	Pediapred	Non-reactive
Prednisolone	Prelone	Non-reactive
Pregabalin	Lyrica	Positive for Gabapentin
Procaine	Novocain	Non-reactive
Progesterol		Non-reactive
Promethazine	Allerfen	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Anergan	Positive for Tricyclic Antidepressants (TCA)

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Promethazine	Antinaus	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Aprobit	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Avomine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Baymethazine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Diprozin	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Fenergan	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Methazine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Phenergen	Positive for Tricyclic Antidepressants (TCA)
Propranolol	Beta-Progane	Non-reactive
Propranolol	Inderal	Non-reactive
Propranolol	Inderetic	Non-reactive
Propranolol	Inderex	Non-reactive
Pseudoephedrine	Afrinol	Non-reactive
Pseudoephedrine	Sudafed	Non-reactive
Pseudoephedrine	Tylenol Cold (combination)	Non-reactive
Ranitidine	Pylorid, Zantac	Non-reactive
Risperidone	Risperdal M-TAB, Risperdal Consta, and Risperdal	Positive for Fentanyl
Salbutamol	Aerocrom	Non-reactive
Salbutamol	Aerolin	Non-reactive
Salbutamol	Airomir	Non-reactive
Salbutamol	Asmasal	Non-reactive
Salbutamol	Combivent	Non-reactive
Salbutamol	Duovent	Non-reactive
Salbutamol	Ventide	Non-reactive
Salbutamol	Ventodisks	Non-reactive
Salbutamol	Venotlin	Non-reactive
Salbutamol	Volmax	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS (Conc.at 100,000 ng/ml)
Secobarbital	Seconal	Positive for Barbiturates (BAR)
Selegiline	Eldepryl	Positive for Methamphetamine (MET) and Amphetamine (AMP)
Sennosides	Senokot	Non-reactive
Sertraline	Zoloft	Non-reactive
Sodium Valproate	Depakene	Non-reactive
Sodium Valproate	Depakote	Positive for Kratom
Sodium Valproate	Epilim	Non-reactive
Temazepam	Restoril	Positive for Benzodiazepines (BZO)
Testosterone	Andropatch	Non-reactive
Testosterone	Restandol	Non-reactive
Testosterone	Sustanon	Non-reactive
Testosterone	Viormone	Non-reactive
Testosterone	slang name "anabolic steroids"	Non-reactive
Thioridazine Hydrochloride	Mellaril	Non-reactive
Thyroxine Sodium	Eltroxin	Non-reactive
Tramadol	Ultram	Positive for Tramadol (TRA)
Tramadol	Tramake	Positive for Tramadol (TRA)
Tramadol	Zamadol	Positive for Tramadol (TRA)
Tramadol	Zydol	Positive for Tramadol (TRA)
Trazodone	Desyrel	Non-reactive
Trazodone	Molipaxin	Non-reactive
Trazodone	Trialodine	Non-reactive
Triazolam	Halcion	Positive for Benzodiazepines (BZO)
Venlafaxine	Effexor	Non-reactive
Venlafaxine	Effexor XL	Non-reactive
Warfarin Sodium	Coumadin	Non-reactive
Warfarin Sodium	Marevan	Non-reactive