# microlife SureFit+™ Blood Pressure Cuff

## For Microlife automatic upper arm blood pressure monitors.\*

Thank you for purchasing an accessory cuff. This cuff is suitable for arm circumferences of 8.7" to 18.1" to fit most adult, and is easy to adapt to most units.

### Identify your cuff tube connector

Determine if your cuff tube connector, found on the end of the tube connected to the cuff, is all black or yellow tipped.

#### Cuff tube connector – black (standard)

Simply replace your existing Microlife cuff with the SureFit+ cuff. Once the cuff is properly connected to the monitor, prepare the cuff for your measurement.

#### Cuff tube connector – yellow or blue

If the cuff port on your monitor is yellow or blue, please follow these steps to replace the connector:

- Remove the black connector from your SureFit+™ Cuff tube.
- Remove the yellow or blue connector from your existing Microlife cuff tube.
- Insert the yellow or blue connector into the tube on your SureFit+™ cuff.
- Connect the cuff to the monitor by placing the yellow or blue connector into the color coordinating port on the left side of your monitor, following your monitor's instructions. Yellow connectors to yellow ports, blue connectors to blue ports. Do not otherwise combine the colored pieces as they are not interchangeable. Using the wrong cuff connector may cause inaccurate readings.

**Please contact customer service if your connector is a different color than listed as additional steps are needed to ensure an accurate reading.**

### Fitting the cuff

- The cuff is pre-formed for easier use. Remove tight or bulky clothing from your upper arm.
- Position the cuff on the arm as illustrated, so that the tube is closer to your lower arm. Make certain that the lower edge of the cuff, with the red edge, lies approximately 1/2" (1 to 2 cm) above the elbow and that the tube is closer to the inner side of the arm.

**IMPORTANT:** The red strip on the edge of the cuff (Artery Mark) must lie over the artery which runs down the inner side of the arm.








**TIP:** Align red artery mark to pinky finger.

- To secure the cuff, wrap it around your arm and press the hook and loop material together. Make sure that the size indicator arrow falls on the green "OK" range when the cuff is in position. If it doesn't, you are using the wrong cuff size.
- There should be little free space between the arm and the cuff. You should be able to fit 2 fingers between your arm and the cuff. Clothing must not restrict the arm. Any piece of clothing which does must be removed. Cuffs that don't fit properly result in false measurement values. Measure your arm circumference if you are not sure of proper fit.
- Lay your arm on a table so the cuff is at the same height as your heart. Make sure the tube is not kinked.
- Remain seated quietly for two minutes before you begin the measurement.

### Comment

If it is not possible to fit the cuff to your left arm, it can also be placed on your right arm. However, all measurements should be made using the same arm.

Comparable blood pressure measurements always require the same conditions (relax for two minutes before a reading). Remain still during your measurement, do not flex muscles.

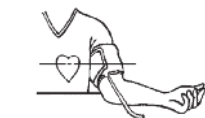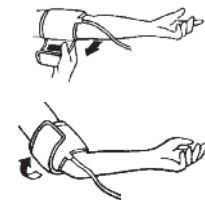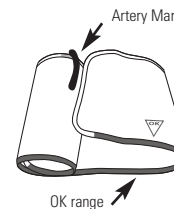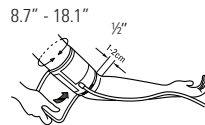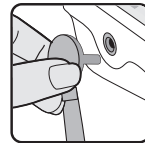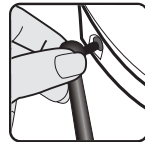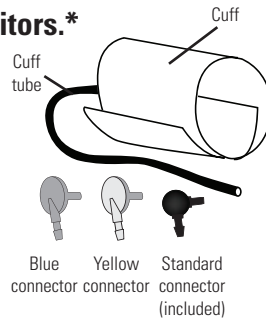### Caution

**Be sure to consult the instruction booklet included with your monitor for correct use of this cuff with your blood pressure monitor. For use only with Microlife automatic blood pressure monitors. Not compatible with semi-automatic monitors, or any other company's blood pressure monitors.**

- Proper blood pressure cuff size and placement is essential to the accuracy of the blood pressure reading. See arm cuff chart on the outer packaging for sizing information.
- Wrapping the cuff too loosely (preventing proper inflation) may result in inaccurate blood pressure readings.

### Care and maintenance

- The cuff contains a sensitive airtight bubble. Handle this cuff carefully and avoid all types of stress through twisting or buckling.
- Clean the device with a soft, dry cloth. Do not use gasoline, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. The cuff must not be washed in a dishwasher, clothes washer or submerged in water.
- Handle the tube carefully. Do not pull on it. Do not allow the tubing to kink and keep it away from sharp edges.
- Do not use the device if you think it is damaged or if anything appears unusual.

### Limited Warranty

Your Sure-Fit+ cuff is warranted for one year by Microlife USA Inc, against manufacturer defects for the original purchaser only, from date of purchase.

The warranty does not apply to consequential and incidental damages, or damage caused by improper handling, and accidents. Professional use, not following the operating instructions, and alterations made to the monitor or accessory by third parties, are also not included in this warranty. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

Microlife USA Inc will investigate your concern. A cuff as defined by this warranty, determined to be out of specification, will be replaced and shipped to you at no cost. A cuff as defined by this warranty, determined to be within specification, will be returned to you with a report of findings, at no cost.

Please use the below customer service contact information to reach Microlife USA Inc. regarding any warranty concerns. We ask that you please contact us before sending any product back in order to better identify, and more quickly process, your concern.

### Technical specifications

Cuff type: ..... SureFit+, Wide range cuff for arms circumference 8.7"-18.1", 22-46 cm

Weight: ..... 160 g

Cuff pressure display range: .. 0-299 mmHg

Storage temperature: ..... -20 to +55°C (-4° to +131°F)

Operation temperature: ..... 10 to 40°C (50° to 104°F)

Operation/storage humidity: .. 15 to 90% relative humidity maximum

Technical alterations reserved.

Made in China

### How to contact us

Microlife USA, Inc.

1617 Gulf to Bay Blvd., 2nd Floor, Clearwater, FL 33755

Toll Free Help Line: 1-800-568-4147, Fax: (727) 451-0492

Email: [custserv@microlifeusa.com](mailto:custserv@microlifeusa.com), [www.microlifeusa.com](http://www.microlifeusa.com)

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

**Not for use on children 12 years of age or younger.**

**Questions? Call the Customer Help Line Toll Free 1-800-568-4147.**

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