



REORDER NUMBER:

**RTG-02500**

**Trach O-Life™**

**Tracheostomy Care Kit**

**One Compartment Tray**

**Includes:**

**1 Pair Vinyl Gloves, 1 Water Proof Drape**



**1 Twill Tape, 1 Tracheostomy Dressing**

**4 Gauze Sponges 4" X 4", 1 Clear Brush**

**2 Pipe Cleaners, 2 Cotton Tip Applicators**

## RxOnly

Caution: Federal law (USA) restricts this device to use by or at the direction of a physician

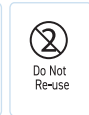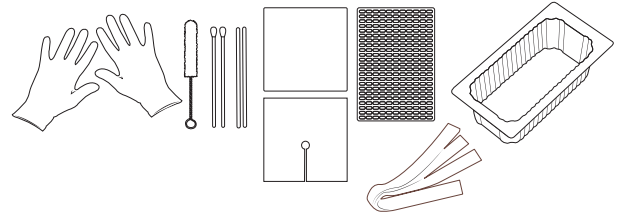

Clean, Ready to Use  
For Single Patient Use

## INSTRUCTIONS FOR USE:

1. Wash hands with soap and water.
2. Open trach care kit and spread waterproof drape on to the work area.
3. Pour 3% hydrogen peroxide (not included) into the tray and sterile water in cup (not included).
4. Place glove on dominant hand.
5. Remove items from kit with dominant hand and put them on waterproof drape.
6. Suction patient.
7. Disconnect patient from vent with NON-dominant hand.
8. If using a non-disposable inner cannula unlock and remove inner cannula with NON-dominant hand and place it in tray with 3% hydrogen peroxide (not included) to soak.
9. Insert a clean inner cannula into trach tube and reconnect patient to vent.
10. Use included brush to clean inner cannula.
11. Rinse inner cannula with sterile water.
12. Use a 4x4 gauze sponge to dry inner cannula.
13. Store clean inner cannula per facility procedures & guidelines.
14. Suction patient as needed.
15. Remove soiled 4x4 gauze sponge from patient using non-dominant hand.
16. Clean skin around stoma area with a sterile water moistened gauze sponge.  
Dry skin around stoma area with a clean 4x4 gauze sponge.
17. Apply a clean 4x4 trach dressing around stoma and change tube holder if needed.
18. Discard all soiled disposables per facility procedures & guidelines.
19. Wash hands with soap and water.

## WARNING:

Do not resterilize, may result in infection or allergic reaction. Avoid freezing and excessive heat.

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F

For more info

Telephone  
| 323-761-0035

E-mail  
| info@rtghealth.com

[www.rtghealth.com](http://www.rtghealth.com)