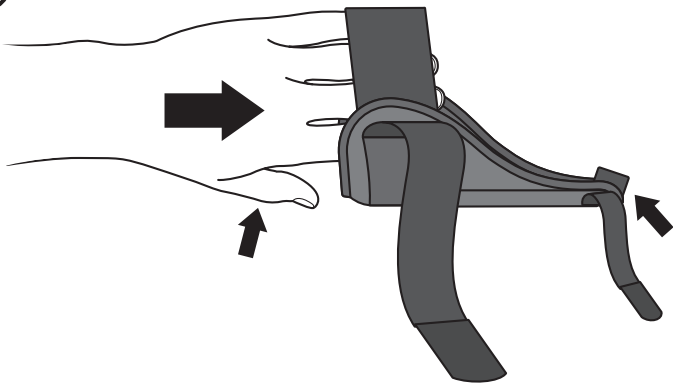
1

Example illustrates left hand application. Splint can be worn on either hand. Slide affected hand through wrist band and into thumb orthosis. Be sure thumb and thumb strap at the front of the orthosis are aligned.

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2


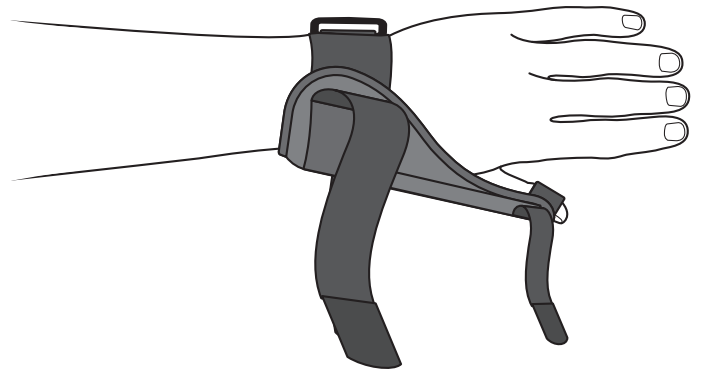
Illustration shows hand and thumb properly positioned in the orthosis.

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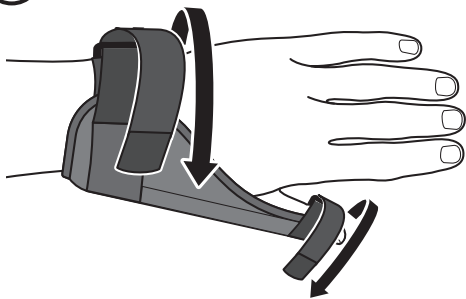3


Thread wrist strap through D-ring and pull the strap snug but not too tight. Attach strap end to thumb orthosis. Then wrap the thumb strap around the thumb as shown.

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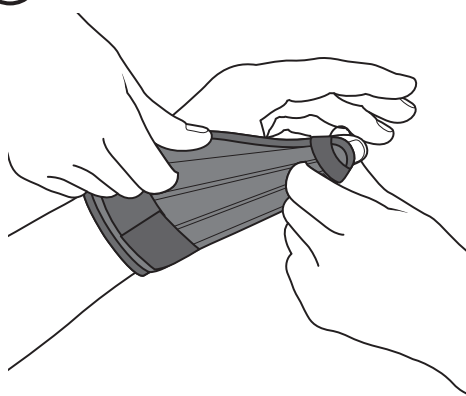4

For improved fit and comfort, the metal stays can be molded around the thumb by bending the malleable material.

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Thumb Support Instructions for Use

Férula para pulgar
Instrucciones de Uso
Support de pouce
Mode d'utilisation

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 USA
P: 1-800-321-0607 | +1-760-795-5440
F: +1-760-795-5295
www.breg.com

WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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